# One Beam Bilirubinometer





## One Beam

One Beam is GINEVRI's new concept bilirubinometer designed to achieve accurate bilirubin serum level measurements in newborns through the analysis of micro-blood samples.

Knowledge of accurate bilirubin levels is essential helping in the successful treatment of neonatal jaundice caused by bilirubin aggregation in various body tissues, skin in particular (classic yellow skin colour observed in jaundice patients). Bilirubin aggregation could involve brain tissues causing irreversible patient damage (kernicterus). Newborns require constant monitoring of bilirubin levels. Through its ease of use and remarkable accuracy One Beam provides required support to doctors.

One Beam allows for a prompt photometric analysis of bilirubin in the serum (conjugated and non-conjugated total) using a capillary tube as an optical cell.

Bilirubin concentration is determined through a photometric measurement at 455nm and 575 nm wavelengths: the first wave length indicates bilirubin levels while the second informs on any interfering agent presence, essentially haemoglobin.

A mathematical model accounts for haemoglobin interference resulting in an accurate bilirubin result.

### The HD OLED display indicates both bilirubin (mg/dl and $\mu$ mol/l) and interfering agent (g/dl) values.

One Beam uses a single optical beam projected onto a fixed point of the sample due to a particular automatic filtering system. The system achieves a "clean" light beam free of any disturbing frequencies reducing negative bilirubin photo-isomerization effects thus ensuring a stable signal.

An accurate signal is also achieved as a only one point on the capillary tube is measured eliminating any errors observed in past generation devices with two measure points.

Lastly a stronger signal results due to the use of a single rather than a split beam.

A particular collimation and beam concentration system allows for reading results even with limited serum sample quantities obtained after centrifugation, as in polycythemia cases (up to 80% haematocritic levels) or partial capillary tube filling.

Advanced microprocessor controlled electronic technology enables better signal processing transformed into the corresponding bilirubin level promptly indicated on the display. An optional printer enables quick printing of any test results.

#### MAIN CHARACTERISTICS

Ease of use: complete operation consists in taking a blood sample from the newborn (less than 55µl-two drops!), place it into the capillary tube then centrifuging for a period of 5 minutes at 12.000 rpm.

**Disposable single-use capillary tube**: an heparinized capillary tube of a 55µl or less volume is used as a disposable single-use cell avoiding the need for unsafe, expensive or breakable cuvettes and other special test tubes.

Quick-test: testing is promptly executed once the centrifuged capillary tube is inserted into the One Beam.

Total self-correction: the microcontroller detects and corrects any interfering agent presence.

#### REFERENCES

- Developed together with the Department of Sensors of Tor Vergata University (Rome).

- Tested by UOC of Pediatric, Neonatology and TIN of the "S.Giovanni Calibita" Fatebenefratelli General Hospital (Rome).



#### Consumables

11412B73) Restabil Standard Value, 2 High and 2 Low 11144A73) Heparinized capillary tubes, pack of 1.000 pcs 569) Sealing wax for 1.000 capillary tubes 6442) Thermal printing paper, 1 pc 12957A73) Lancets, pack of 10.000 pcs V05-038575) STERIHEEL PREMATURE Lancets, pack of 200 pcs V05-031025) STERIHEEL NEWBORN Lancets, pack of 200 pcs

#### TECHNICAL SPECIFICATIONS

USF SAMPLE VOLUME CUVETTE UNIT OF MEASUREMENT TYPE OF MEASUREMENT INTERFERENCE AVERAGE READING TIME MAX. RESOLUTION MAX. ACCURACY SENSOR **OPTICAL FILTERS RESULTS OUTPUT** ADDITIONAL FUNCTIONS DIMENSIONS WEIGHT POWER SUPPLY

Measurement of total bilirubin levels in newborns Centrifuged blood Less than 55 µl Heparinized glass capillary tubes 4/30 mg/dl or 68/510 µmol/l Photometric Automatic correction 2 s +/- 0.1 mg/dl o +/- 0.1 µmol/l +/- 1% (FS+ mis.) Silicon photodiode 455 and 575 nm OLED display, PC, Printer (optional) Date and Time display 15x22x24 cm (WxDxH) 2 Kgs (2,3 Kgs with printer) 230 Vac, 50/60 Hz, 15VA (25VA with printer)



Tel.: +39 06 93 459 330 Fax: +39 06 93 459 393 e-mail: export@ginevri.com www.ginevri.com Quality System



The specifications in this catalogue are indicative. The company GINEVRI srl reserves the right to make changes, without further notice, to the products described within this catalogue in order to improve reliability, function or design.

ISO 13485:2012

ISO 9001:2008